

Customer Information Bulletin

Rocket[®] Seldinger, & Seldinger SG & PK Kits

Dear Customer

Our current design of our seldinger kit introducer needle has served us well over the last several years.

However, in recent months we have had suggestions and requests for improved performance, including the ability to have increased visibility under ultrasound.

At Rocket, we take customer feedback very seriously and as a result we have developed and tested new design concepts, and have sourced new suppliers to meet the new design requirements.

The introducer needle has changed from a Tuohy needle and is now a straight Seldinger needle as shown.

- The use of a straight needle will allow easier insertion through to the pleural space.
- The silicone ring can be used to illustrate how deep the needle was inserted before aspiration
- The measurement provided by using the silicone ring can be used to set the position of the sliding stop on each dilator to suit the size of the patient and help prevent over insertion of the dilators.
- The Echogenic tip will allow increased visibility of the tip under ultrasound.

The needle will be available in all the Seldinger kits.

Customer Information Bulletins on all our products can be found on our website at:

http://www.rocketmedical.com/Rocket_support_CIB_main.htm

If you have any questions, please contact your nearest Customer Services Team for assistance:

Customer Services

Rocket Medical plc
Sedling Road. WASHINGTON. Tyne & Wear. NE38 9BZ. England
Tel: +44 (0) 191 419 6988
Fax: +44 (0) 191 419 6989

