

# Rocket<sup>®</sup> Rectal Solution Set

The Rocket<sup>®</sup> Rectal Solution Set has been developed in conjunction with a leading UK Spinal Injuries Unit to provide a simpler and safer method of injecting rectal solutions to aid bowel movement in patients with neurogenic bowel dysfunction (e.g. spinal cord injury).

The syringe rings give a larger, easier grip which, when combined with the soft rectal nozzle, provide an easier and safer method of administration than using small syringes alone.

The set is suitable for self-administration of a rectal solution either by the patient or personal carer for conditions where normal bowel function is compromised.

Use only on the advice of your doctor, nurse specialist, community nurse or continence advisor who will assess your suitability for this device.



- Wash your hands. Open the pack and remove the syringe and nozzle assembly.
- Attach the nozzle to the syringe by gently screwing the white moulding into the top of the syringe.
- Locate and turn clockwise until lightly tightened.
- Draw up the required dose of your rectal injection solution. The rectal injector can also be used to improve the administration of micro-enemas.
- KY Jelly<sup>®</sup> may be used to lubricate the nozzle if required.
- Using the most convenient position (e.g. standing, sitting on the toilet or lying on your side with your knees drawn up), gently insert the nozzle into the rectum 3-4cm. Slowly depress the plunger of the syringe to expel the contents into the bowel, wait a few seconds, then withdraw the nozzle completely.
- Discard the combined syringe and nozzle assembly and wash your hands.
- See pack leaflet for full Instructions for Use.

Ordering Information	Code
<b>Rocket<sup>®</sup> Rectal Solution Set</b> for the administration of rectal solutions Supplied individually wrapped in cartons of 10 units	<b>R56215</b>

